RESCUE LIFE

Manual and semiautomatic (AED) defibrillator with monitor

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RESCUE LIFE

The combined defibrillator monitor Rescue Life is designed to be used by out-of-hospital and hospital users. Having an innovative and fully featured design, it provides for defibrillation and monitoring in a single portable device.

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DISPLAY

Brilliant, high contrast, wide-angle TFT LCD colour display allows the simultaneous visualization of up to 3 ECG channels. Information such as heart rate, SpO2 value, alarms and operational functions are clearly visible on the display even in low visibility conditions. In AED or Advisory mode, text messages and voice prompts guide the operator during the CPR operations.

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FAST CONNECTION

Users can choose between ergonomic reusable paddles, suitable for the manual defibrillation in both adult and paediatric patients or disposable pads for AED defibrillation. Both types are equipped with a single fast lock connection for reliable rescue operations. The ECG can be detected by 3,5 or 10 leads ECG cable and the optional SpO2 sensor is available for detecting blood oxygen saturation. Rescue Life can be equipped with internal reusable defibrillation paddles with interchangeable different sizes.

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ERGONOMIC DESIGN

Rescue Life is impact proof, compact and lightweight. The dedicated comfortable and washable carrying bag keeps all the accessories at hand and protects Rescue Life during the emergency procedures. The case is built with washable materials and has a comfortable handle for safe transportation. The paddles have dedicated seat on the back of Rescue Life with a fast hook-unhook procedure. The IPX4 standard permits the operation under adverse climatic situations.

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PROFESSIONAL SOLUTIONS

Rescue Life includes manual, AED and Advisory operation modes allowing the best use in different situations. It is designed to be used by doctors and trained individuals. The manual mode provides very easy three steps operation: energy selection from 1-230J, energy charge and shock. In AED mode, the voice prompts and display messages are assisting the operator during CPR operations, charging automatically to 150J when a shock-able ECG rhythm is detected. In Advisory mode, the system alerts when the shock is necessary (using voice prompts and display messages) leaving the energy level and charging choice to the operator.

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MEMORY AND DATA MANAGEMENT

Simple commands, extremely intuitive and easy to use. Immediate access to main setup parameters using only the trim knob and function keys. The internal memory is able to record more than 300 hours of ECG data and events. The data are recallable and printable on the integrated thermal printer. Printing can be done in manual or in automatic mode. In the automatic mode the printing initiates when the energy charging starts (recording the ECG signal before the shock) and will continue 10 seconds after the shock was delivered.

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Technology for diagnosis and emergency

• DESIGN

Born from expert Italian designers, its strength is simplicity and beauty. Attractive line, ergonomic form, studied in every detail, Rescue Life is the first professional defibrillator totally MADE IN ITALY that respects the traditions of elegance and comfort. Thanks to the lightweight and transport facility this defibrillator is the perfect device in the hard challenges. Quality and safety is the mix of our expertise, applied to Rescue Life as well to all our products line.

• TECHNOLOGY BIPHASIC

The Rescue Life defibrillator uses a (BTE) BIPHASIC truncated exponential waveform with the HiCap technology which brings the benefits of current controlled systems but with higher reliability. Controlling the waveform duration based on the body impedance measurement, the Rescue Life is able to maintain the ideal duration, peak current and voltage of the defibrillation shock. The optimal biphasic waveform adopted by Rescue Life provides the confidence of the first shock success without the need for escalating energy.

• USABILITY

Rescue Life is designed to be the ideal partner of the protagonists in the hardest emergency situations. Since the beginning, our goal was not to replace the man by the machine, but to create a product able to support the operator in every in and out-of-hospital situations. The clear graphics and intuitive interface allow an easy use even by a less experienced user. The excellence of the monitor and the reliability of the defibrillator gives the user a ALL IN ONE emergency.

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Technical Specifications

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GENERAL SPECIFICATIONS

Dimensions	● 369 x 240 x 340 mm
Weight	● 5,5 Kg with battery
Electrical	● AC 100-240 VAC, 50/60 Hz - inverter 12V car adapter (optional) ● Rechargeable 14.4 V - 2 Ah NiMH battery
Capacity	● 100 shocks at 230J with battery fully charged
Software	● Voice and texts multilanguage
Memory	● Internal flash 2 Gb up to 300 hrs of ECG data, events and usage review

ENVIRONMENTAL

Temperature	● Operational -5° ~ 55° C ● Storage -20° ~ 70° C ● Relative humidity 10 ~ 95%
Isolation	● ECG type CF, Defibrillation type BF
Water proof	● Class IPX4

DEFIBRILLATION

Waveform	● Biphasic Truncated Exponential (BTE) HiCAP technology (Large storage capacitor)
Impedance	● Compensated, range 25 - 200 Ohm
Charge time	● 6 sec for 230 J with battery fully charged

AED MODE (OPTIONAL)

Energy	● Fixed energy at 150 J
Protocol	● AHA 2010 CPR Guidelines with voice and text prompts
Shockable rhythms	● VF with amplitude > 100 mV and VT with rhythm >150 bpm
Sensitivity	● Shockable rhythm-VF > 95% ● Shockable rhythm-VT > 75%
Specificity	● According to AHA 2010 ● Normal sinusoidal rhythm > 99% ● Asystole and other non-shockable rhythms > 95%

MANUAL MODE

Energy range	● 1 - 230 J (from 1-10 J in 1 J steps; from 10 to 230 J in 10 J steps)
Commands	● Multifunction trim knob. Charge and shock button directly in the front panel for hands free defibrillation
Paddles	● Reusable adult & pediatric defibrillation pads with charge/shock command ● Disposable defibrillation pads and internal defibrillation pads (optional)
Operating mode	● ECG « R » wave synchro or asynchro mode
Indicators	● Battery and main led indicators ● Clear and visible backlight color buttons

DISPLAY

Type	● High contrast TFT color LCD 5.7" (320 x 240 pixels)
Information	● Operational information, ECG traces, SpO2 value, NIBP values, parameters, Heart Rate indicator, filters, alarms and battery status

ECG MONITORING

Bandwidth	● 0.5 - 120 Hz (-3 db) with filters off
CMRR	● > 90 db
Input impedance	● > 20 Mohm
Input	● Patient cable 3 leads (I,II,III) - standard ● Patient cable 5 leads (I,II,III,aVL,aVR,aVF) - optional ● Patient cable 10 leads (12 channels ECG) - optional
Gain	● 5, 10, 20 mm/mV with patient cable. AUTO with pads
Velocity	● 5, 10, 25, 50 mm/sec
Filters	● 50/60 Hz, EMG Filter, base line
HR range	● 20 - 300 bpm (~ 2%), digital readout on the display
Alarm	● HR max settable 250 bpm; HR min 20 bpm

SPO2 (OPTIONAL)

SpO2 range	● 0 - 100%
HR range	● 18 - 300 ppm
Accuracy	● 70 - 100% ~ 2% for adults with finger clip sensor
Alarm	● Adjustable min 65%

PRINTER

Type	● Integrated thermal printer for ECG traces and events documentation hardcopy including HR/SpO2/NIBP values
Paper speed	● 5, 10, 25 mm/sec
Paper width	● 58 mm
Operating model	● Manual, automatic (10" pre and post shock recording)

EXTERNAL PACEMAKER (OPTIONAL)

Type	● Rectangular wave
Operating mode	● Fixed on demand
Pulse rate	● 30 ppm to 250 ppm, adjustable in steps of 5 ppm
Impulse duration	● 22 ms
Pulse current	● 0 to 150 mA adjustable in steps of 5mA
Amplitude	● Max 150 V

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Module NIBP / Module PNI Rescue Life

The NIBP option available on **Rescue Life** is focused on *Advantage A+* module that provides the highest NIBP performance in the smallest complete package available.

The module is designed to meet the needs of most demanding clinical applications thanks to its reliability and flexibility by providing the highest level of patient motion tolerant performance also during any movement by ambulance or helicopter transport.

TECHNICAL SPECIFICATIONS

Technique: Oscillometric

NIBP Accuracy: Meets ANSI/AAMI SP10-2002, EN 1060-4

Patient Application: adult/paediatric/neonatal

Systolic Range: Adult: 40-260 mmHG, Paediatric: 40-160 mmHG
Neonatal: 40-130 mmHG

Range MAP: Adult: 26-220 mmHG, paediatric: 26-133 mmHG
Neonatal: 26-110 mmHG

Diastolic Range: Adult : 20-200 mmHG, paediatric: 20-120 mmHG
Neonatal: 20-100 mmHG

PR RANGE: 30-220 BPM

PR Accuracy : +/- 2% or +/- 3 bpm

Transducer Accuracy : +/- 3 mmHg over full range in operating conditions

Operating Modes: Manual, Long Term Automatic, Stat, Service

Operating Conditions: 0°C - 50°C, 15% - 95%, non condensing humidity

Storage Conditions: -20°C - 65°C, 15% - 90%, non condensing humidity

Auto Interval Periods: 1,2,3,4,5,10,15,30,60 and 90 minutes

Safety & Regulatory Standards

IEC60601-1, IEC/EN60601-2-30, AAMI SP10, OIML R 16-2

EN1060-1, EN1060-3, EN1060-4

Avantages

SMT (Standard Motion Tolerance): Standard Motion tolerance which provides a better accuracy for the patient monitoring.

TMT (Transport Motion Tolerance): perfect for ambulance and helicopter EMS services.

HDM (Haemodialysis Monitoring): special accuracy to provide reliable NIBP for renal disease patients.

RMT (R-wave Motion Tolerance): all motion and noise artefacts are quickly rejected to isolate arterial pulses and provide reliable NIBP in the most challenging patient motion condition.

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